



Sling Specialist in Safe Patient Mobilization

Sling Cleaning Instructions

Alpha Modalities LLC reusable , wipeable and ECO product cleaning instructions are listed below. Failure to clean per our guidelines will void product warranty.

Washable & ECO Line

- Commercial Laundry Systems with maximum load of 850lbs - tunnel washer voids warranty
- Oxygen based bleach solutions only - chlorine based solution will void warranty
- Fresh water rinse cycle of minimum eight (8) minutes
- Maximum water temp of 185F / 85C
- Maximum dry temp of 145F or tumble dry low heat

Wipeable Line (SW)

- Wipe with hospital approved disinfectant - follow manufacturers dwell times or
- 10% Bleach solution
- Do Not Launder

Do not dry clean or iron any products

Always inspect sling before use, if damaged or torn, please contact www.alphamodalities.com and click on support

