

AM-TSL-L






Item/Color	Size	SWL
AM-TSL-L	46" x 86"	1000 Lb/ 454 Kg
AM-TSL-W	50" x 86"	1000 Lb/ 454 Kg

SLING PLACEMENT ON BED

- ▶ MAY PLACE A CHUCK OR SINGLE SHEET BETWEEN PATIENT AND SLING.
- ▶ FOOT LABEL SHOULD BE AT FOOT END.
- ▶ PLACE PILLOW BETWEEN LEGS FOR KNEE COMFORT.

LOG ROLLING

- ▶ CHOOSE THE TWO STRAPS CLOSEST TO PATIENT'S MIDDLE AREA- (TYPICALLY 2 & 3).
- ▶ USE THE PURPLE LOOP TO ATTACH SLING.

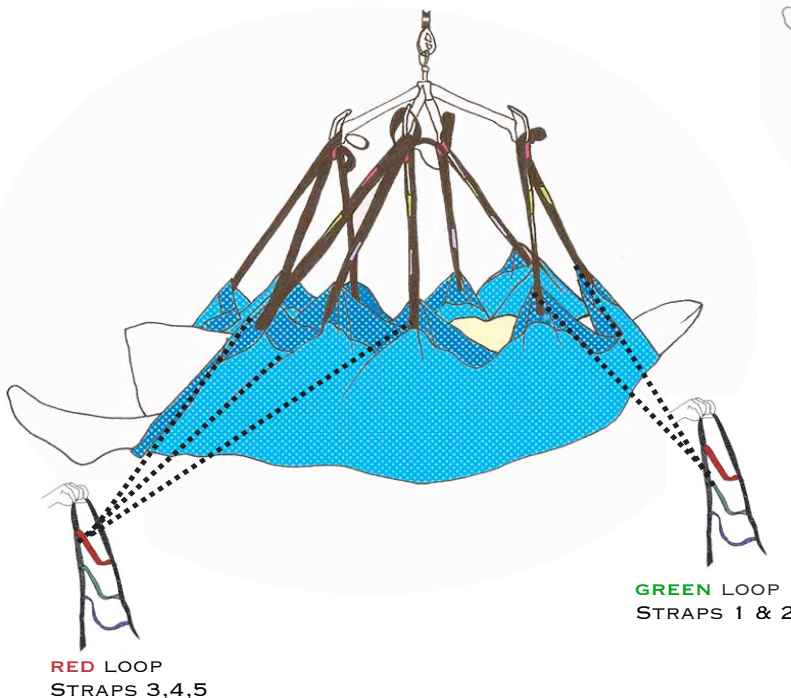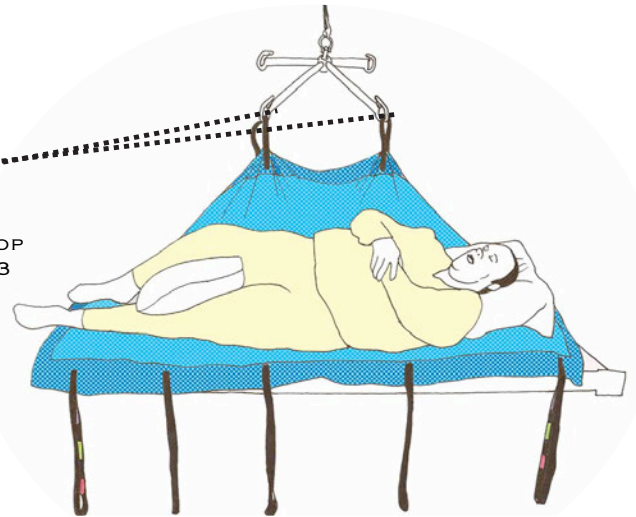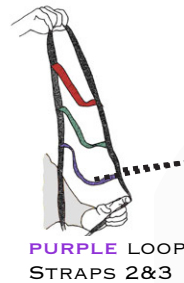


SUPINE TRANSFER

- ▶ ATTACH STRAPS 1-2 ON BOTH SIDES TO THE 2 HOOKS NEAREST TO PATIENT'S HEAD.
- ▶ ATTACH STRAPS 3, 4 & 5 ON EITHER SIDE TO THE 2 HOOKS NEAREST TO FEET.


RECLINED TRANSFER

- ▶ ATTACH STRAPS 1 & 2 ON BOTH SIDES TO THE 2 HOOKS NEAREST TO PATIENT'S HEAD.
- ▶ ATTACH STRAPS 3 & 4 ON EITHER SIDE TO THE 2 HOOKS NEAREST TO FEET.
- ▶ DO NOT ATTACH STRAP 5.

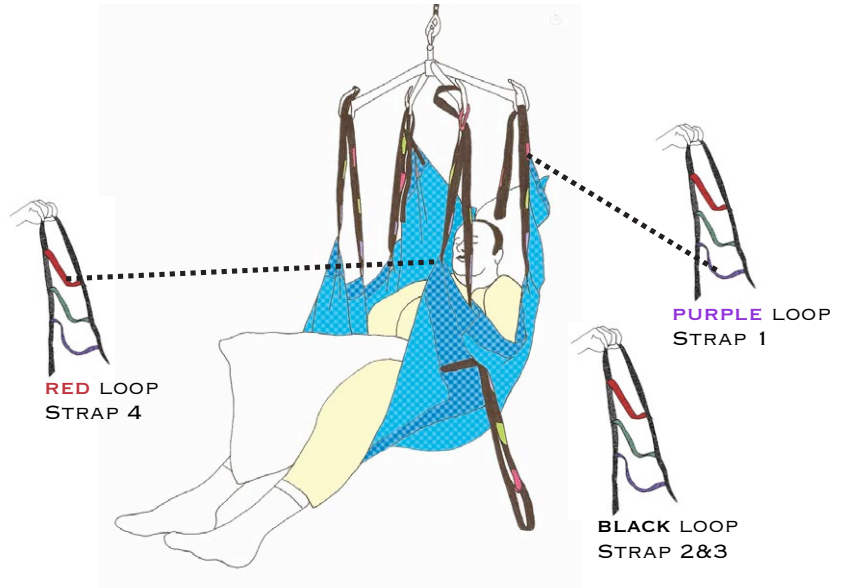

BARIATRIC PATIENTS OVER 600 LB
USE AM-TSL-W AND THE DUAL HOIST CONFIGURATION AS SHOWN BELOW.

SUPINE TRANSFER

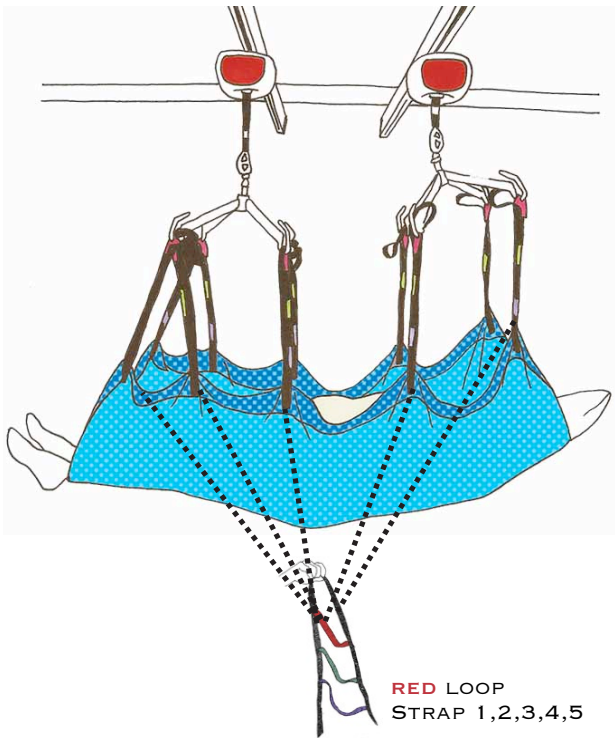

RECLINED TRANSFER

