




PURPLE LOOP
STRAPS 2&3

LOGROLLING


CHOOSE THE TWO STRAPS
CLOSEST TO THE PATIENT'S
MIDDLE AREA

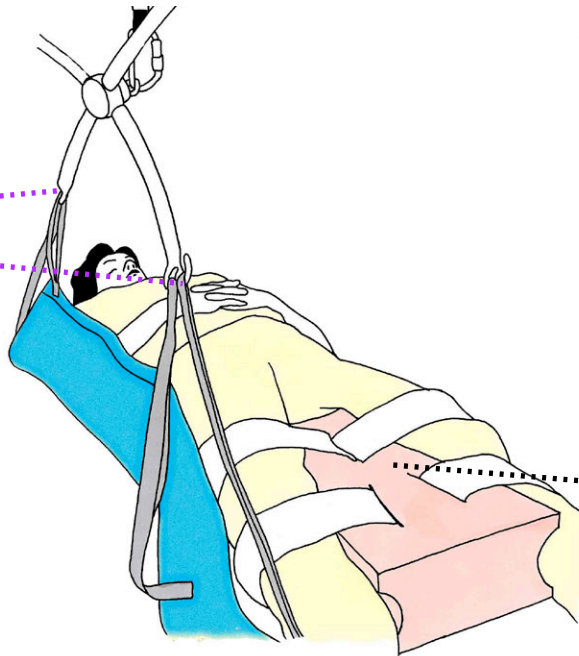

USE ABDUCTOR WEDGE
BEFORE LOG ROLLING
PATIENT

SUPINE TRANSFER

- PLACE ABDUCTOR WEDGE BETWEEN LEGS.
- SECURE BEFORE REPOSITIONING OR LIFTING

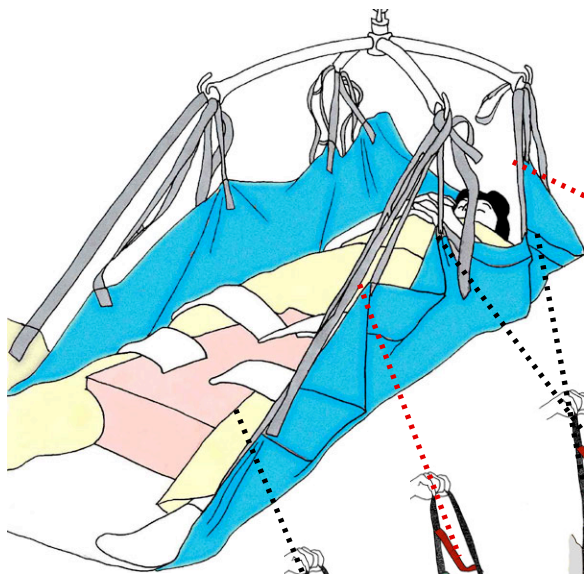

STRAP 1
RED LOOP




STRAP 2 & 3
BLACK LOOP



STRAP 4
RED LOOP



STRAP 5
BLACK LOOP

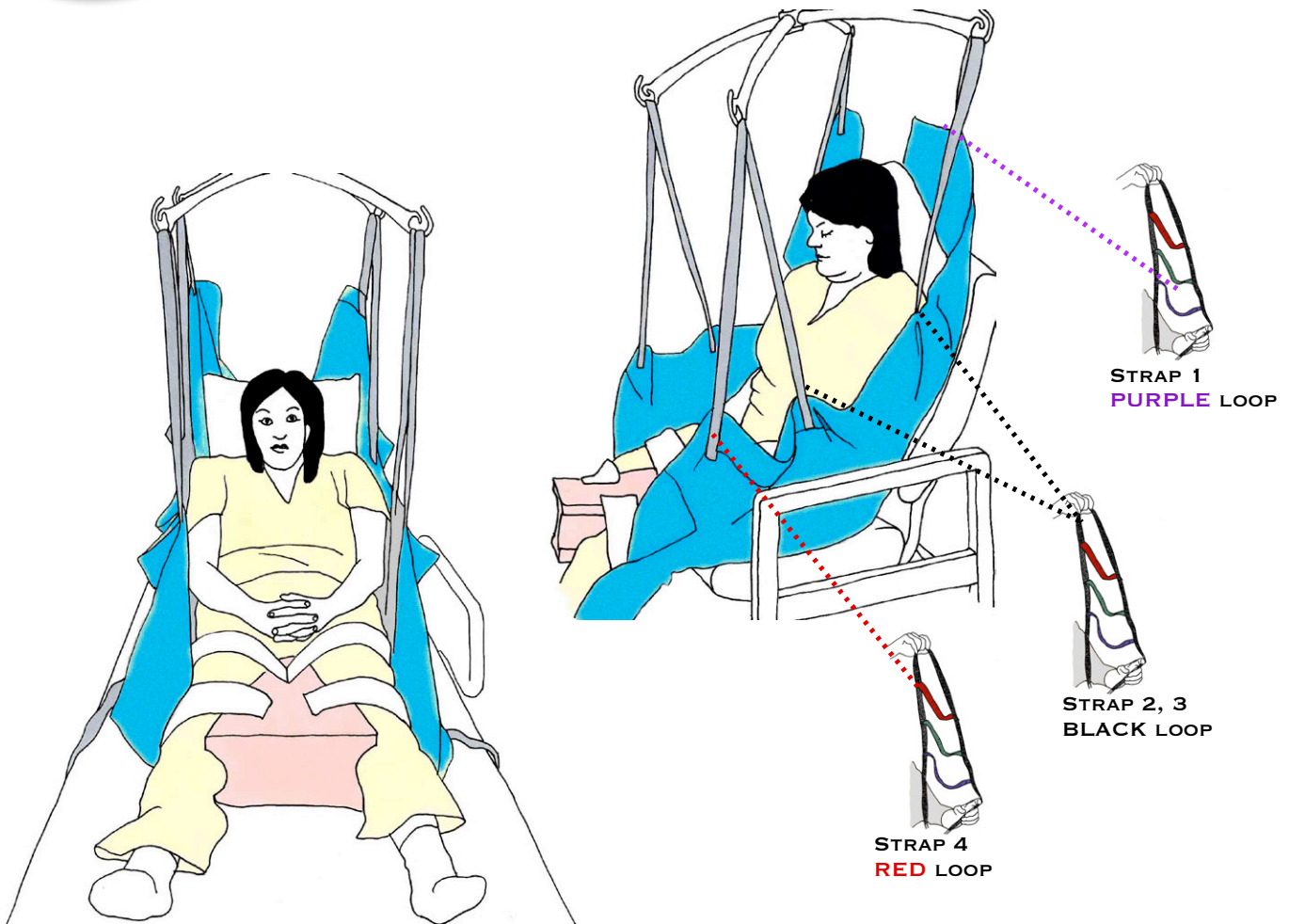

TRANSFER TO GERI-CHAIR

- PLACE ABDUCTOR WEDGE BETWEEN LEGS AND SECURE- ATTACH LOOPS TO LIFT PATIENT IN SEATED POSITION.
- WHEN LOWERING PATIENT INTO CHAIR, SUPPORT THE ABDUCTOR WEDGE WITH A HAND UNTIL PATIENT IS FULLY SEATED- THEN REMOVE.

