




1. Have patient lean slightly forward and place sling down to patient's lower back.
2. Weave strap through clip and adjust inner safety belt to patient waist. Tighten strap as tight as possible without causing discomfort.



Check with nursing if new surgical wounds are located near belt.

3. Bring SARA® 3000 mobile lift close to patient. Attach black clips to Sara® 3000:
 - Choose clip depending on patient's ability to bear weight. **The closer the clip, the more support that is offered to the patient.**
 - As patient is being raised, the belt may need to be tightened to offer more support.

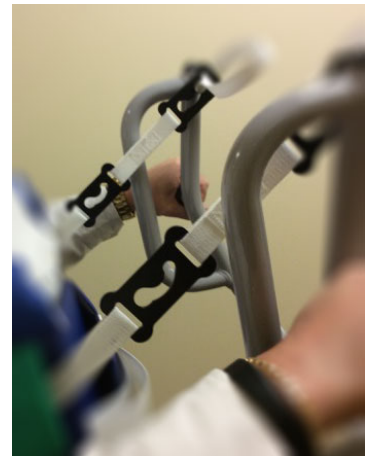

Size/Color Tab	Waist Range*	SWL
M	36" - 50"	300 KG/ 660 LB
L	40" - 57"	
X	44" - 69"	

*if in doubt, step down one size.

Cleaning:
Wipe with approved hospital disinfectant between patients. Do not launder.

