



| Item | Chest Size | SWL |
|------------------|--------------------|----------------|
| A-WalkVSWWL-Peds | 20-26" / 50-66cm | 660 Lb/ 300 Kg |
| A-WalkVSWWL-S | 34-39" / 86-97cm | |
| A-WalkVSWWL-M | 39-43" / 99-109cm | |
| A-WalkVSWWL-L | 44-48" / 112-122cm | |
| A-WalkVSWWL-XL | 49-53" / 124-135cm | |

Placing WalkVest

Use only a single layer of clothing or gown- remove all other layers before placement.




Place vest around patient’s chest while in seated position.



- Adjust the top strap and secure the buckle
- Adjust the bottom strap and secure the buckle



Hold both straps by the loops and pull outward to tighten. The vest should be tight without causing discomfort to the patient.

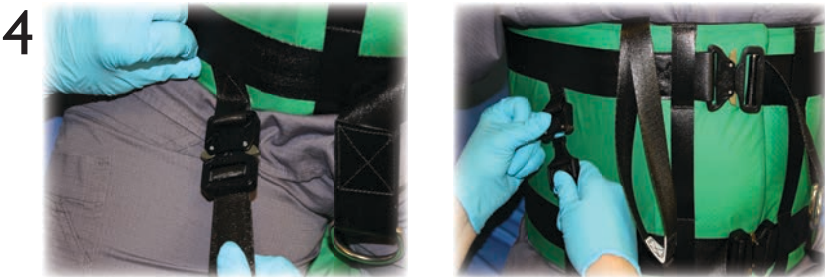

- Gently bring the leg straps underneath and up between the patient’s legs.
- Based upon the size of the upper thigh of the patient, choose either the lower buckle or upper buckle and secure each leg strap. *Do not criss cross the straps.*

Attach Walk Vest to hanger bar

4pt Hanger Bar:



Attach each of the four straps to each one of the hooks on the hanger bar.

2pt Hanger Bar:



Attach the left side straps to the left side hook.
Attach the right side straps to the right side hook.

Cleaning Instructions

- Wipe with approved hospital disinfectant between patients
- DO NOT soak in chlorine based solution
- DO NOT LAUNDRER