

Alpha
 Modalities

A-SeatSWWL Series w/
 A-Amputee Option



Size/Trim Color	Recommended Weight Range
A-SeatSWWL-S	31-75 lb / 14-34 kg
A-SeatSWWL-M	75-145 lb / 34-66 kg
A-SeatSWWL-L	145-220 lb / 66-100 kg
A-SeatSWWL-X	220-440 lb / 100-200 kg
A-SeatSWWL-XX	440-800 lb / 200-363kg

***Note: Chest straps, leg straps, and amputee straps are now black and not white.**

1 Place sling behind patient. Slide down to coccyx of the patient. Take each leg piece and run through the double dignity straps-underneath patient stumps.



2 Pull one leg strap up along the **outside** of one side of sling and feed through the amputee loop.



Pull the other leg strap **inside** the opposite side of the sling and feed into amputee loop.



3 Attach straps to hanger bar

For seated position: a slight recline is recommended. **Do not** attach shortest loop for chest straps. **Do** use long loop for leg straps.



For semi-reclined transfer: Long loop for chest straps and short loop for leg straps.



Once patient is suspended, move to commode, chair, wheelchair, etc...

Cleaning Instructions

- Wipe with approved hospital disinfectant between patients.
- DO NOT soak in chlorine based bleach solution
- DO NOT LAUNDR