

Alpha Modalities

A-BariVest Series



Size/Trim Color	Recommended Girth Ranges	SWL
A-BariVest-PED	24"-38"	880lbs/ 400kg
A-BariVest-S	46"-66"	
A-BariVest-M	54"-74"	
A-BariVest-L	75"-94"	
A-BariVest-X	89"-109"	

Disclaimer:

- Care team must accompany patient
- Patient must be able to bear partial weight
- Patient should use a walker to assist
- Recommend using a 4 pt hanger bar



1

Logroll patient onto vest. The smaller flaps should be down towards the patient's legs (*the label should be down towards the patient's feet*) Straps should be facing down when sling is unfolded and beneath patient. Match the center rectangle with buttocks.



2

Once patient lies flat on sling, pull lower flaps over legs. Elevate head of bed to sit patient up.



3

Fold lower flap over abdomen.



4

Clip sides of vest and tighten straps as tight as possible without causing discomfort.

Alpha Modalities

A-BariVest Series

6



Dangle patient's feet at edge of bed to prepare to egress. Attach straps to hanger bar. **Ensure that whichever loop used on the left side of the patient is the same loop used on the right side.**

7



Hanger bar positioning:

- 2pt hanger bar should be positioned *behind* patient's head.
- 4pt hanger bar should be positioned *above* patient's head.

As patient stands, weight will settle into vest.

DUAL HOIST CONFIGURATION

Use when patient weighs more than 600 Lbs

- Use one hanger bar on each side of the patient and attach straps to corresponding hanger bar.
- Raise hanger bars on either side. Make sure they end at the same height.
- As patient stands, weight will settle into vest. We recommend a walker to assist ambulation.



Cleaning Instructions:



A-BariV

Launderable

- Wash Temp not to exceed 185F / 85C
- Drying - Medium Heat, not to exceed 165F / 74C
- Oxygen Based Bleach Recommended
- Max Load 850 lbs Cycle



A-BariVSWWL

Wipeable

- Wipe with approved hospital disinfectant between patients.
- DO NOT soak in chlorine based bleach solution
- DO NOT LAUNDR