

**A-Limb**

SWL 150kg /330lb



A-LimbTWF8



A-LimbSWWL



A-LimbSPU

### Placing Limb Sling for use

*If your sling has no pocket, skip to step 3*

*Note: A-LimbSWWL and A-LimbSPU are not made with a pocket*



1 Place hand into the pocket of Limb Sling with **palm facing out**.



2 Drape Limb Sling strap over the pocket in order to ensure label and strap are face down.




3 Gently push the Limb Sling into the mattress and under the patient's limb.

**Label and strap SHOULD NOT touch the patient.**



4 Pull Limb Sling up from the other side of patient's limb. Pull up straps and attach to hanger bar as you see fit.

**Note: The closer the strap loop is to the sling, the quicker the patient's limb will raise.**

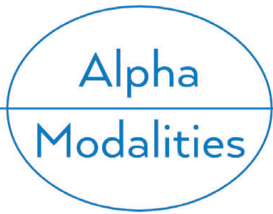

**A-Limb**

SWL 1000 LB /454 KG

## Variations of Use

*For all Limb Slings*



Place sling above or below the knee to raise limb for placing pillows underneath, cleaning, and/or care.

Side-lying position to raise limb for cleaning and/or care.


## Cleaning Instructions



- Wash Temp not to exceed 185F / 85C
- Drying - Medium Heat, not to exceed 165F /74C
- Oxygen Based Bleach Recommended
- Max Load 850 lbs Cycle



- Wipe with approved hospital disinfectant between patients.
- DO NOT soak in chlorine based bleach solution
- DO NOT LAUNDRER

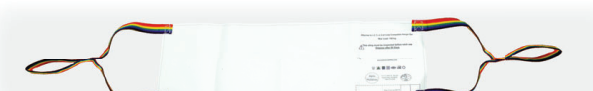

- Single Patient Use only
- Dispose of sling if soiled or patient no longer is using it.
- Reprocessing - contact Alpha Modalities for details