

A-Pannus

SWL 454kg / 1000lb




Place soft disposable chuck on leading edge of Pannus sling and place underneath fold. Encourage patient to assist in pulling pannus forward.



Head of bed must be elevated **a min of 20 degrees** & patient must be able to tolerate pannus being moved forward and shifted onto their body.



Securely attach lower straps to bed frame. We suggest you **wrap the straps with padding**, such as a towel or pillow case to prevent contact with patient's skin.

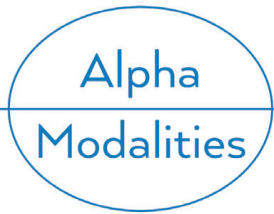

A-Pannus

SWL 450kg / 1000lb

USING THE SLING

Option A

Caregiver on either side pulling in the direction of headboard, patient pulling on their pannus and clip onto headboard



Option B

Use Ceiling Lift or Mobile Lift. Hook up shortest loop on strap and raise hanger bar (pulls up & forward). Caregiver will need to gauge how far up to raise the pannus and what the patient can tolerate.



Cleaning Instructions

- Wash temp not to exceed 185F/ 85C
- Drying- medium heat, not to exceed 165F/ 74C
- Oxygen based bleach recommended
- Max load 850lbs cycle