

A-SeatTW

Size/ Trim Color	Recommended Weight range
A-SeatTW-S	31-75 lb / 14-34 Kg
A-SeatTW-M	75-145 lb / 34-66 Kg
A-SeatTW-L	145-220 lb / 66-100 Kg
A-SeatTW-X	220-440 lb / 100-200 Kg
A-SeatTW-XX	440-800 lb / 200-363kg



Placing Seat Sling for Use

1

- Elevate head of the bed and place sling behind patient with *label side down*. The leg wings should come along the sides of patient.
- Lean patient forward and slide center of sling down to the coccyx area.

OR



If head of bed cannot be elevated:

- Roll patient to one side and gently push the sling into the mattress and under the patient.
- Roll patient to other side and pull sling out and spread out under patient. Return patient to their back.



- Pull each leg wing underneath patient's legs just above the knee.
- Take left leg wing and pull through dignity strap located on the right leg wing.
- Pull leg straps to straighten fabric under patient.

Transferring patient with the seat sling



Attach straps to hanger bar:

- FOR SEATED POSITION - Short loop for chest straps and long loop for leg straps.
- FOR SEMI-RECLINED TRANSFER - Long loop for chest straps and short loop for leg straps.



After lifting, move patient by holding the sling or using one of the handles located on the back of the sling.



- *For caregiver in front of patient:*
Lower patient into chair by pushing gently on the leg wings.
- *For caregiver behind patient:*
Use the handle to ensure patient's buttocks are in contact with the rear of the chair before lowering.

NOTE: While lowering patient, make sure their back is in constant contact with the chair.

Cleaning Instructions

- Wash Temp not to exceed 185F / 85C
- Drying - Medium Heat, not to exceed 165F / 74C
- Oxygen Based Bleach Recommended
- Max Load 850 lbs Cycle