
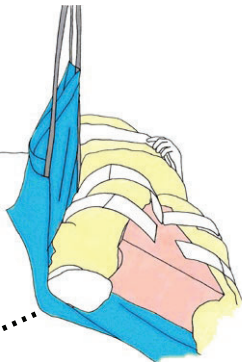
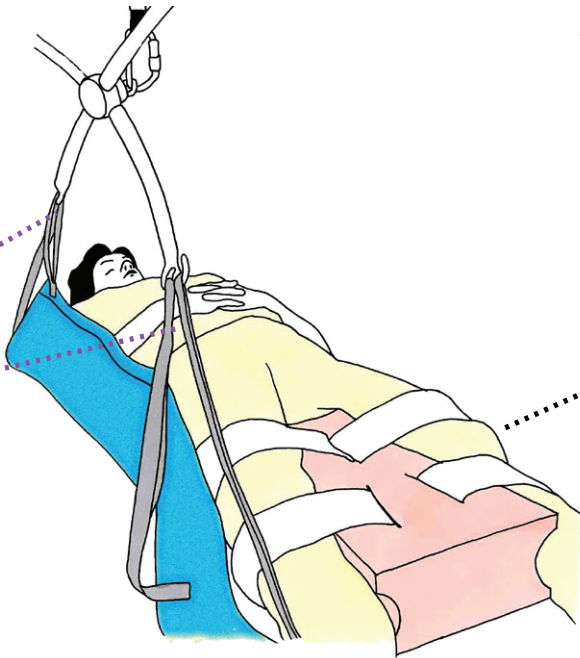


A-TSL Series
(Hip Trauma)

Size	SWL
46" x 86"	1000 Lb/ 454 Kg
50" x 86"	1000 Lb/ 454 Kg

LOGROLLING



CHOOSE THE TWO STRAPS CLOSEST TO THE PATIENT’S MIDDLE AREA


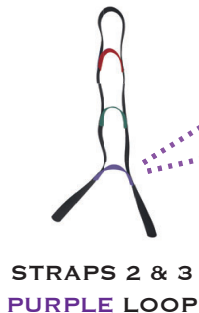


USE ABDUCTOR WEDGE BEFORE LOG ROLLING PATIENT

SUPINE TRANSFER

- PLACE ABDUCTOR WEDGE BETWEEN LEGS
 - SECURE BEFORE REPOSITIONING OR LIFTING

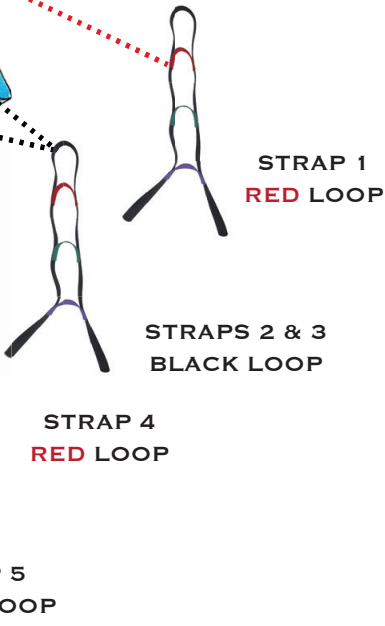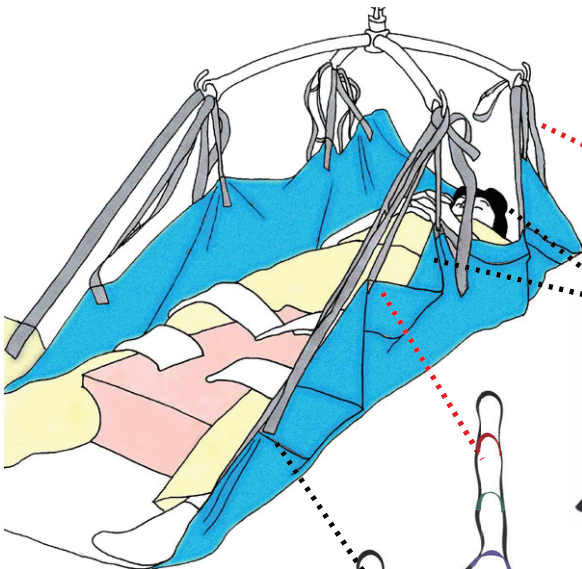

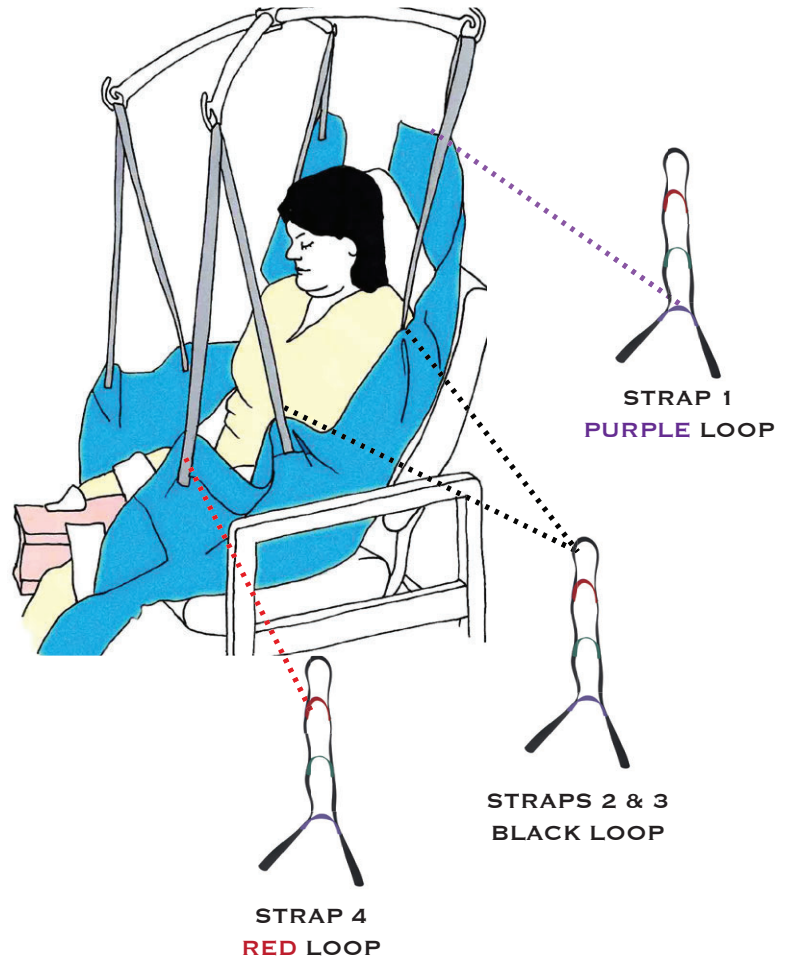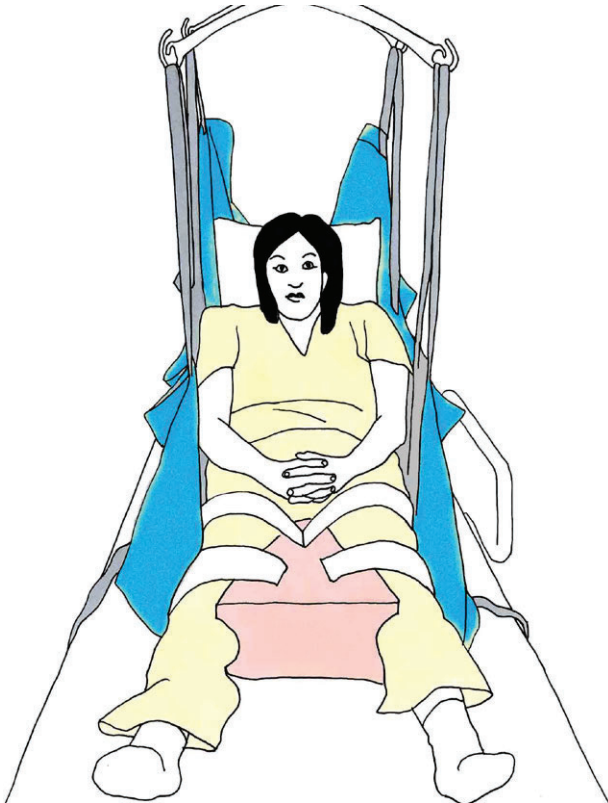

TRANSFER TO GERI-CHAIR

- PLACE ABDUCTOR WEDGE BETWEEN LEGS AND SECURE - ATTACH LOOPS TO LIFT PATIENT IN SEATED POSITION
- WHEN LOWERING PATIENT INTO CHAIR, SUPPORT THE ABDUCTOR WEDGE WITH A HAND UNTIL PATIENT IS FULLY SEATED - THEN REMOVE

