


A-TSLCL-Series
(4pt hanger bar)

SIZE	SWL
46" × 86 "	1000 Lb/ 454 Kg
50" × 86"	1000 Lb/ 454 Kg

SLING PLACEMENT
ON BED

- If needed, a chuck may be placed between patient and sling.
- Foot label should be at patient's foot.
- Place pillow between legs for knee comfort.

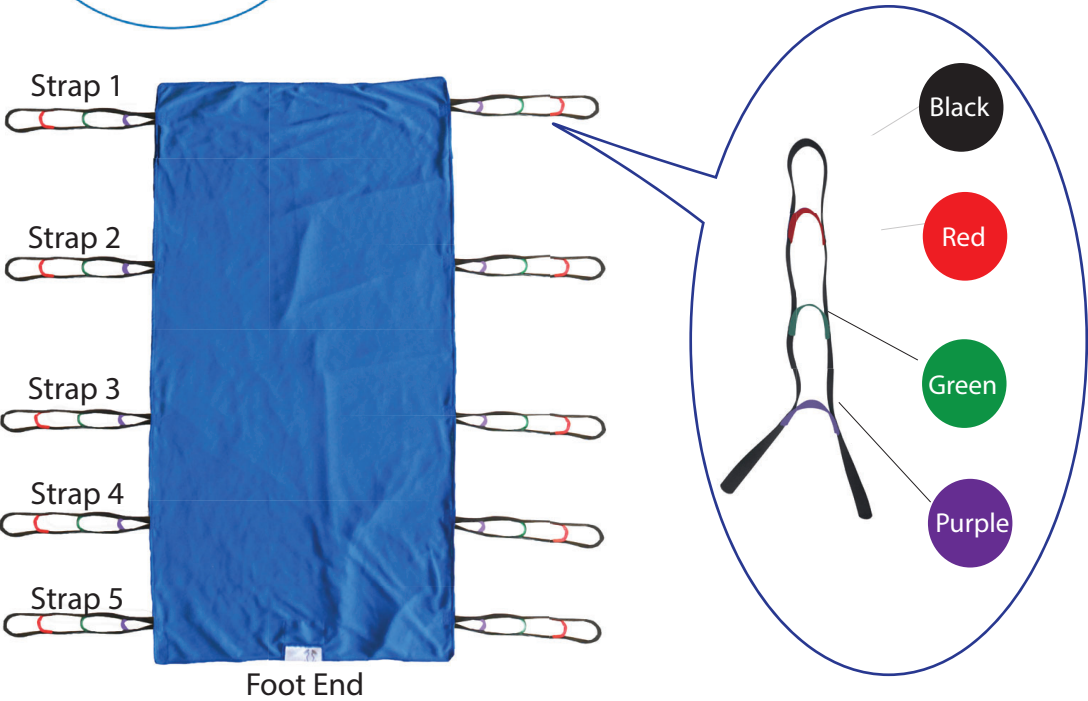

NOTE: The loop color attachments shown are a *general guideline*. Please adhere to your hospital's policy and use your clinical judgment as to what best meets your patient's needs.

USING THE TSL TO TURN PATIENT

Attach two straps closest to *patient's middle area*: close to patient's shoulder and hip



A-TSLCL-Series
(4pt hanger bar)

SUPINE TRANSFER

Generally use shorter loop at the head compared to the rest of the body



CARDIAC CHAIR TRANSFER

Attach strap 5 if patient tolerates leg hyperextension - otherwise, leave strap unattached



Cleaning Instructions

- Wash Temp not to exceed 185F / 85C
- Drying - Medium Heat, not to exceed 165F / 74C
- Oxygen Based Bleach Recommended
- Max Load 850lbs Cycle

DISCLAIMERS:

- Discontinue use if loose stitching or tears are visible.
- Do not use with an uncleared C-Spine.