


Alpha
 Modalities

**A-TSLF8-Series
 (4pt hanger bar)**

SIZE	SWL
46" x 86"	1000 Lb/ 454 Kg
50" x 86"	1000 Lb/ 454 Kg

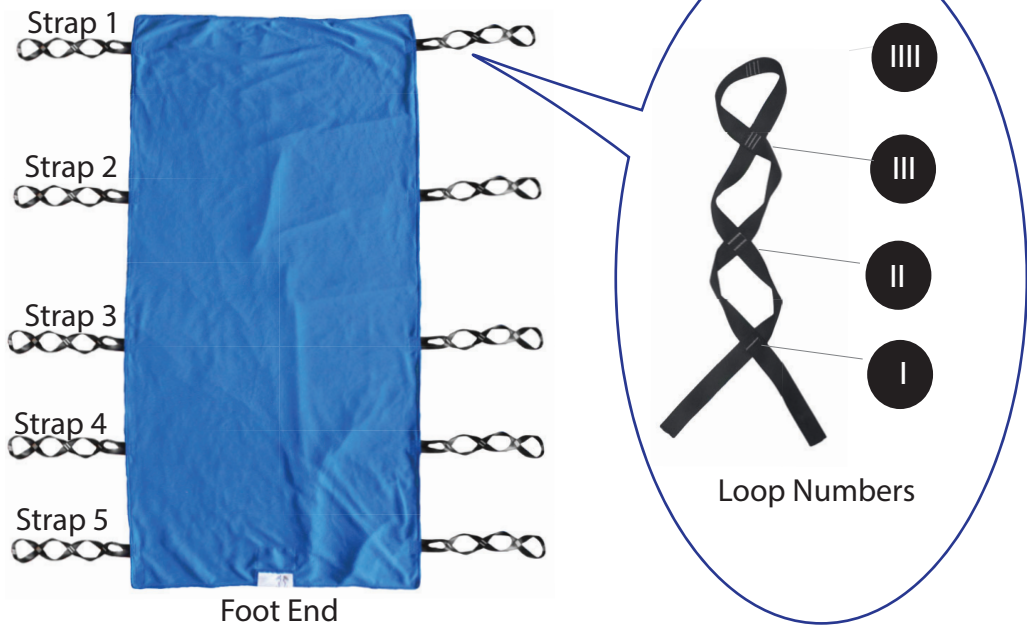

- SLING PLACEMENT ON BED**
- If needed, a chuck may be placed between patient and sling.
 - Foot label should be at patient's foot.
 - Place pillow between legs for knee comfort.

NOTE: The loop number attachments shown are a *general guideline*. Please adhere to your hospital's policy and use your clinical judgment as to what best meets your patient's needs.


USING THE TSL TO TURN PATIENT

Attach two straps closest to *patient's middle area*: close to patient's shoulder and hip



SUPINE TRANSFER

Generally use shorter loop at the head compared to the rest of the body

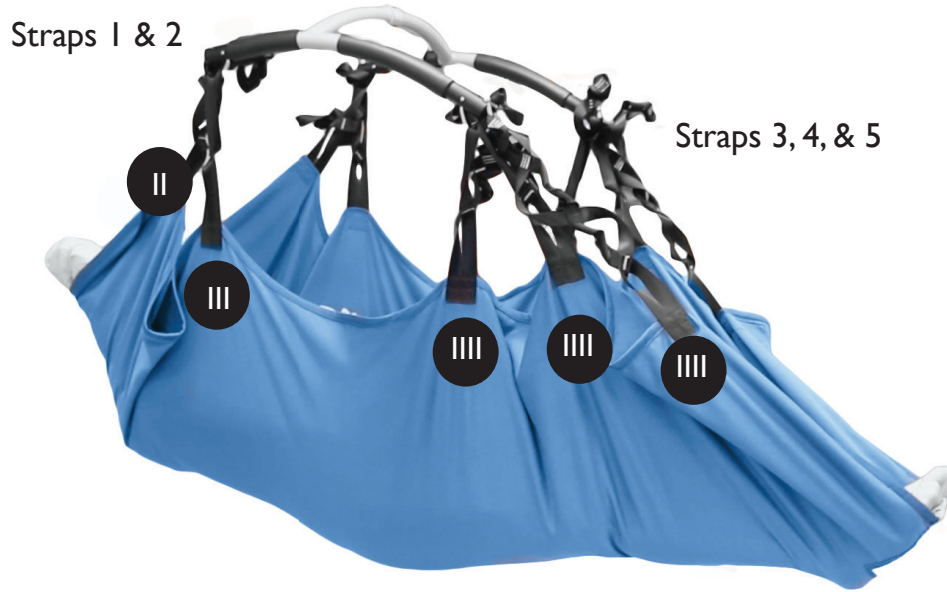

INCLINED TRANSFER

Attach strap 5 if patient tolerates leg hyperextension - otherwise, leave strap unattached



Cleaning Instructions

- Wash Temp not to exceed 185F / 85C
- Drying - Medium Heat, not to exceed 165F / 74C
- Oxygen Based Bleach Recommended
- Max Load 850lbs Cycle

DISCLAIMERS:

- Discontinue use if loose stitching or tears are visible.
- Do not use with an uncleared C-Spine.