

A-TTurner

SWL 330 LB / 150 KG



Placing TTurner under patient



Lay TTurner on bed
with label side up



Roll up approximately
one quarter of the way.



Roll patient to one side and
place TTurner under patient's
back with the roll (and label)
facing down.




Roll patient to
the other side.



Unroll TTurner.
Attach strap securely to bed
frame on the side you will be
turning patient towards.

Attaching TTurner strap to patient's bed

- A - Choose an opening at the base of the bed frame.
- B - Push strap through the opening
- C - Pull strap up around frame and connect the Velcro pieces, making sure they are firmly attached.

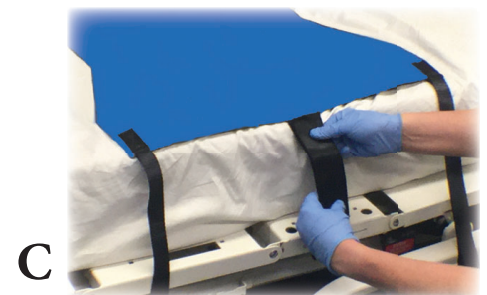

Using TTurner to turn patient



Lower hanger bar. Attach sling loops to hanger bar parallel to the body, opposite the side that is attached to bed frame.

Note: On low ceilings, you may need to lower the bed.



Raise ceiling lift hanger bar. Patient will begin to rise with the sling and turn on their side.

Stop when patient has reached desired position.

Disclaimer: When using TTurner to turn a patient, the railing should be up on the side patient is turning towards.

Cleaning Instructions

- Wash Temp not to exceed 185F / 85C
- Drying - Medium Heat, not to exceed 165F / 74C
- Oxygen Based Bleach Recommended
- Max Load 850lbs Cycle