

### A-SitStandSW Series



Item	Inner Safety Belt
A-SitStandSW-S	29" - 46"
A-SitStandSW-M	38" - 55"
A-SitStandSW-L	45" - 64"
A-SitStandSW-XL	55" - 73"

*\*if in doubt, step down one size.*



1 Have patient lean slightly forward. Place inner belt against article of (end clothing at lower back of patient. **Do not place on bare skin.**




2 Secure inner belt so that it is snug (ends may or may not overlap). Make it secure without causing discomfort to the patient.

3 Weave strap through middle buckle as follows:



Push strap **underneath** middle part of buckle.

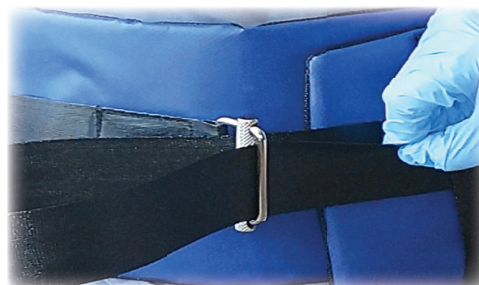


Pull strap back **over** the top of the middle part of buckle.



Tighten strap as tight as possible without causing discomfort.

*Check with nursing if new surgical wounds are located near belt*

- 4 Bring sit stand lift close to patient. Attach sling to lift using either the **A) loops** or **B) clips**, depending on the type of lift you are using. *The closer the loop or clip, the more support that is offered to the patient.*

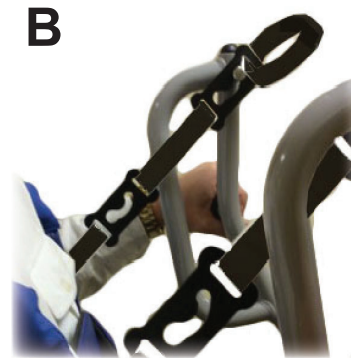

The sling should support the patient but should not pull the patient forward too much. If patient is able, encourage the patient to hold onto the 'support arms'.  
Patient is ready to be lifted.



Try to have the patient assist in raising from chair or keep themselves steady. Operate lift button to raise patient to a height that is comfortable. As patient begins to stand, check inner belt as it may need to be tightened.

#### Cleaning Instructions

- Clean with approved facility disinfectant for "Non Critical Devices" as outlined by FDA
- Do not soak in solution of chlorine based bleach solution
- Do not autoclave or steam clean