

## A-TTurnerBSW

SWL 660 LB / 300KG



### Placing TTurnerBSW under patient



Lay TTurnerBSW on bed with label side up



Roll up approximately one quarter of the way.



Roll patient to one side and place TTurnerBSW under patient's back with the roll (and label) facing down.





Roll patient to the other side



Unroll TTurnerBSW. Attach strap securely to bed frame on the side you will be turning patient towards.

## Attaching TTurnerBSW strap to patient's bed

- A - Choose an opening at the base of the bed frame and push strap through.
- B - Pull strap up and over the top of the middle buckle.
- C - Pull strap down through the bottom of the middle buckle. Strap will now be pointing down.
- D - Grab both ends of straps to tighten.

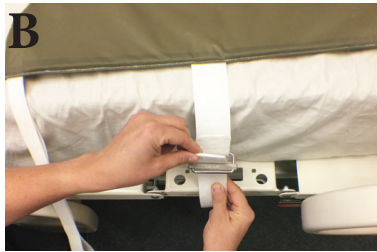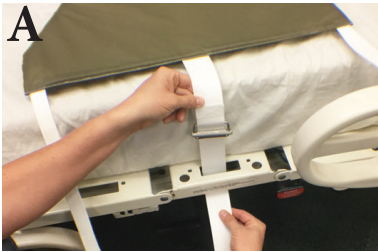

## Using TTurnerBSW to turn patient



Lower hanger bar. Attach sling loops to hanger bar parallel to the body, opposite the side that is attached to bed frame.

Note: On low ceilings, you may need to lower the bed.



Raise ceiling lift hanger bar. Patient will begin to rise with the sling and turn on their side.

Stop when patient has reached desired position.

**Disclaimer:** When using TTurnerBSW to turn a patient, the railing should be up on the side patient is turning towards.

### Cleaning Instructions

- Wipe with approved hospital disinfectant between patients.
- **DO NOT LAUNDER**