

Alpha Modalities

General Sling Inspection Guidelines- Spray Wipeable Slings


Before patient use, *always* do a visual inspection of the sling, check the key areas below, *if compromised, REMOVE FROM SERVICE.*



Ensure the label is intact, readable, and that the weight capacity is clearly marked. *Discontinue use* if unreadable or missing



All of the loops and straps on each side must be free of fraying or cuts. Stitching must be intact.

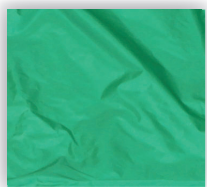

Fabric must be free of worn spots, weave separation or visible cuts. Stitching in seams and panels must also be intact.



800-273-5749

www.alphamodalities.com