



Your Sling Source for Safe Patient Mobilization

1111 South 344th Street, Suite 205, Federal Way, WA 98003 • 800.273.5749 • www.alphamodalities.com

A-TSLSPU
(2pt hanger bar)

Part Number	Size	Max Load
A-TSLSPU	45"W x 86"L	454kg / 1000lb
A-TSLSPUX	55"W x 86"L	500kg / 1100lb

- If needed, a chuck may be placed between patient and sling.
- Label should be at patient's foot.
- Pillow may be placed between legs for comfort.



NOTE: The attachment configurations shown are a *general guideline*. Please adhere to your hospital's policy and use your clinical judgment as to what best meets your patient's needs.

USING THE TSL TO TURN PATIENT

Attach two straps closest to *patient's middle area*: close to patient's *shoulder and hip*



SUPINE TRANSFER

Generally use the same length loop all the way down



SEMI-RECLINED TRANSFER

Generally use the short loop for strap 1 and the longer loop for the rest of the straps to keep the patient in a semi-reclined position. Strap 3 may or may not be needed depending on the patient's body habitus.



The 2 pt hanger bar runs the width of the patient's body

Cleaning Instructions

- **DO NOT LAUNDER** - may be spot cleaned with hot water if soiled by patient.

DISCLAIMERS

- Discontinue use if loose stitching or tears are visible.
- Do not use with an uncleared C-Spine.