


*Your Sling Source for Safe Patient Mobilization*

1111 South 344th Street, Suite 205, Federal Way, WA 98003 • 800.273.5749 • [www.alphamodalities.com](http://www.alphamodalities.com)

**A-TSLSPU  
(4pt hanger bar)**

Part Number	Size	Max Load
A-TSLSPU	45"W x 86"L	454kg / 1000lb
A-TSLSPUX	55"W x 86"L	500kg / 1100lb

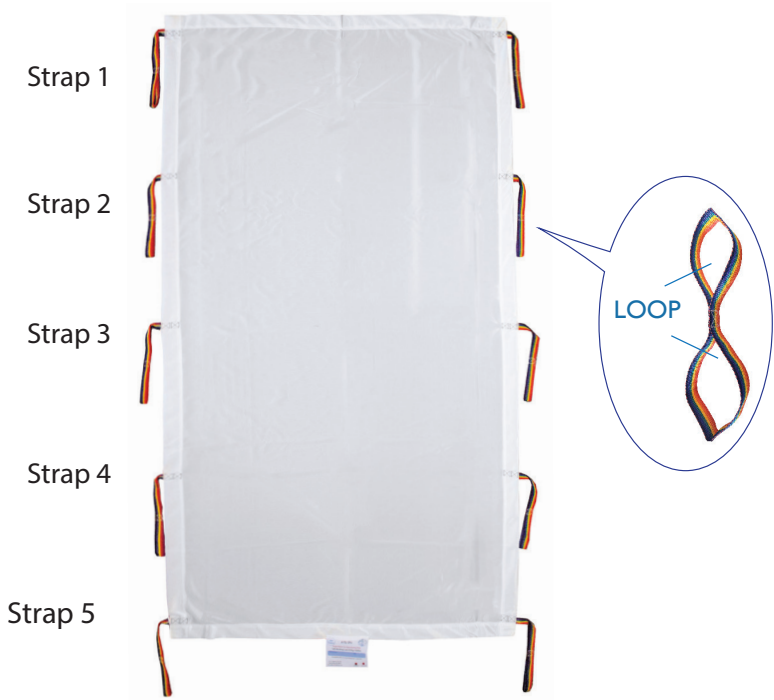

- If needed, a chuck may be placed between patient and sling.
- Label should be at patient's foot.
- Pillow may be placed between legs for comfort.

**NOTE:** The attachment configurations shown are a *general guideline*. Please adhere to your hospital's policy and use your clinical judgment as to what best meets your patient's needs.

**USING THE TSL TO TURN PATIENT**

Attach two straps closest to *patient's middle area*: close to patient's shoulder and hip



## SUPINE TRANSFER

Generally use the same length loop all the way down



## SEMI-RECLINED TRANSFER

Generally use short loop for strap 1 and the longer loop for the rest of the straps  
Attach strap 5 if patient tolerates leg hyperextension - otherwise, leave strap unattached



### Cleaning Instructions

- **DO NOT LAUNDER** - may be spot cleaned with hot water if soiled by patient.

### DISCLAIMERS

- Discontinue use if loose stitching or tears are visible.
- Do not use with an uncleared C-Spine.