


A-WedgeWE Series

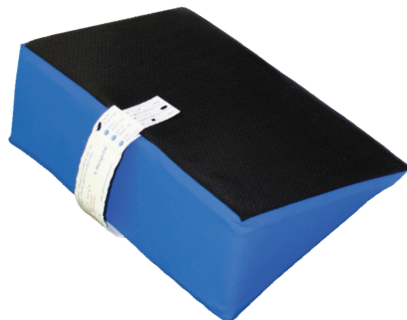

- DO NOT place patient directly on a wedge - use a barrier between the two.
- The wedge should not directly touch the patient's skin.

PLACING THE WEDGE

*Note: **two** wedges are needed for **proper** placement.*




Before placing wedge, turn the patient onto their side using a repositioning sling.

PLACE BOTH WEDGES BEHIND THE PATIENT



Place one wedge **above** and one **below** the sacrum to allow the sacrum to float.
NOTE: Slide your hand under the patient to ensure that the sacral area is off loaded.

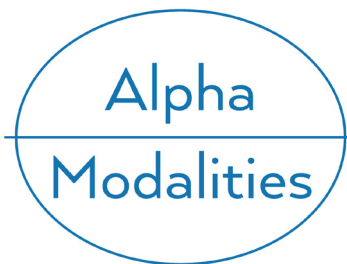

Your Sling Source for Safe Patient Mobilization

1111 South 344th Street, Suite 205, Federal Way, WA 98003 • 800.273.5749 • www.alphamodalities.com


A-WedgeWE Series



Lower patient onto wedges.
Ensure the repositioning sling is
still covering both wedges to
avoid contact with patient's skin.

REMOVING THE WEDGE

Remove patient weight before removing the wedge.

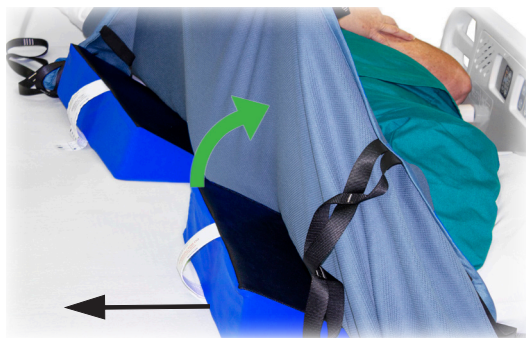

DISCLAIMER: Before removing
wedge, off-load patient weight
from the wedge

Cleaning Instructions- DO NOT THROW AWAY

- **DO NOT LAUNDER** - For soils or stains, use neutral soap suds and lukewarm water. **DO NOT use harsh cleansers, solvents or detergents.** For hard to clean spots, use a soft bristle brush and gentle agitation on troublesome spots or stains. Pre-soak heavy, dried-on soil.
- **Disinfecting** - Quaternary and quaternary/Isopropyl disinfectants are recommended. **DO NOT use phenolic disinfectants.** May be cleaned with a 1:10 dilution of household bleaches containing 5.25% sodium hypochlorite as recommended by the CDC in Atlanta, GA. There is no harmful effect when used in this dilution ratio.
- Disinfectants applied at full concentration or in highly concentrated solutions may damage the fabric
- Iodophor type disinfectants may result in staining.
- **DO NOT DRY CLEAN OR IRON**