

## A-SeatSPU

Size	Max Load
Pediatric/ Small	363kg / 800lbs
Medium/ Large	
Xlarge/ XX	



### Placing Seat Sling for Use



Center and bottom of sling should be at the patient's coccyx area

OR



#### If head of bed *can* be elevated:

- Elevate head of the bed and place sling behind patient with *label side down*. The leg wings should come along the sides of patient.
- Lean patient forward and slide center of sling down to the coccyx area.

#### If head of bed *cannot* be elevated:

- Roll patient to one side and gently push the sling into the mattress and under the patient.
- Roll patient to other side and pull sling out and spread out under patient. Return patient to their back.


2



#### The leg wings should come along the sides of patient.

- Pull each leg wing underneath patient's legs just above the knee.
- Push one leg strap through the bottom loop of the other leg strap.
- Pull strap all the way through - they are now criss-crossed.

#### Amputee Option

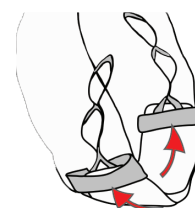

Leg wing/straps slide into the amputee strap on the opposite leg



### Attach straps to hanger bar to transfer patient

FOR SEATED POSITION - Short loop for chest straps and long loop for leg straps.

FOR SEMI-RECLINED TRANSFER - Long loop for chest straps and short loop for leg straps.



### Transferring patient with the seat sling

After lifting, move patient by holding the sling or by using the straps.



#### For transfer into wheel chair:

- *For caregiver in front of patient:*  
Guide patient into chair by pushing gently on the knees.
- *For caregiver behind patient:*  
Use the straps or sling to guide the patient and ensure patient's buttocks are in contact with the rear of the chair before lowering.

**NOTE:** While lowering patient, make sure their back is in constant contact with the chair.