



Your Sling Source for Safe Patient Mobilization

1111 South 344th Street, Suite 205, Federal Way, WA 98003 • 800.273.5749 • www.alphamodalities.com

**A-TSLECO
(4pt hanger bar)**

Part Number	Size	Max Load
A-TSLECO	45"W x 86"L	454kg / 1000lb
A-TSLECOB	55"W x 86"L	500kg / 1100lb

Strap 1

Strap 2

Strap 3

Strap 4

Strap 5



- If needed, a chuck may be placed between patient and sling.
- Label should be at patient's foot.
- Pillow may be placed between legs for comfort.

NOTE: The attachment configurations shown are a *general guideline*. Please adhere to your hospital's policy and use your clinical judgment as to what best meets your patient's needs.

USING THE TSL TO TURN PATIENT

Attach two straps closest to *patient's middle area*: close to patient's shoulder and hip



SUPINE TRANSFER

Generally use the same length loop all the way down



SEMI-RECLINED TRANSFER

Generally use short loop for strap 1 and the longer loop for the rest of the straps
Attach strap 5 if patient tolerates leg hyperextension - otherwise, leave strap unattached



Cleaning Instructions

- Wash temp not to exceed 185F / 85C
- Dry temp not to exceed 145F / 74C
- Oxygen based bleach solutions only
- Max load 850lbs cycle
- 90 Day Warranty

DISCLAIMERS

- Discontinue use if loose stitching or tears are visible.
- Do not use with an uncleared C-Spine.