

*Your Sling Source for Safe Patient Mobilization*

# Alpha Modalities




## 2022/2023 SLING CATALOG

Reusable, Wipeable, SPU, & ECO Series

1111 South 344th Street, Suite 205  
Federal Way, WA 98003  
800.273.5749

[www.alphamodalities.com](http://www.alphamodalities.com)  
[info@alphamodalities.com](mailto:info@alphamodalities.com)



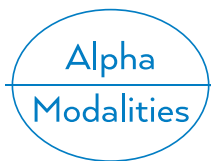

*Your Sling Source for Safe Patient Mobilization*

**Alpha Modalities LLC** is headquartered just outside of Seattle, WA. We began in 1996 with the vision of providing clients with well-integrated Safe Patient Handling solutions that focus on safety, compatibility and cost-effectiveness.


As a *small business*, we have the ability to quickly respond to your needs. We are committed to helping you attain a successful Sling Management Program.

### What makes us **DIFFERENT?**

- **Customized** sling sizing, labeling and colors
- **Quality** training programs
- **Responsive** customer service
- **Individualized** Programs - we work with you to determine which solutions meet your needs
- **Unlimited** 3 year warranty
- **Made in the USA** - All of our materials and products are manufactured in the United States
- **Experience** - We have been the **Sling Specialist** for over 20 years

### Compatibility **Guaranteed**

We design and manufacture washable, wipeable and SPU slings that can be used with all ceiling **and** floor based lifts sold in North America. We test our slings with various manufacturers' hanger bars for compatibility to ensure that they are safe to use and are comfortable for the patient.

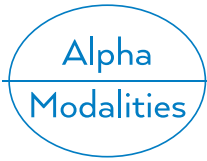

*Your Sling Source for Safe Patient Mobilization*

## **CONTENTS**

### **REPOSITIONING SLINGS**

Page 1.....	A-TSL Series
Page 2.....	A-Limb Series
Page 3.....	A-TTurner Series

### **SEAT SLINGS**

Page 4.....	A-Seat Series
Page 5.....	A-SitOnSling Series
Page 6.....	A-Hygiene

### **SITSTAND BELTS**

Page 7.....	A-SitStandWE Series
-------------	---------------------

### **AMBULATING SLINGS**

Page 8.....	A-WalkVSSWWL
Page 9.....	A-BariV Series

### **SPECIALTY**

Page 10.....	A-PannusIT
--------------	------------

### **LOW TECH**

Page 11.....	A-WedgeSW Series
Page 12.....	A-BedLadderSW

### **SPU/ECO**

Page 13.....	SPU Series
Page 14.....	ECO Series

### **POLICIES**

Page 15.....	Cleaning Instructions
Page 16.....	Warranty Information



## Alpha Modalities

*Your Sling Source for Safe Patient Mobilization*

### A-TSL Series

#### Intended use:

Turns, boosting up in bed, lateral and semi-reclined transfers.

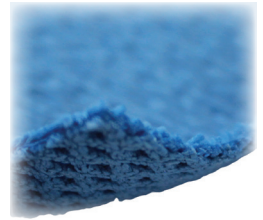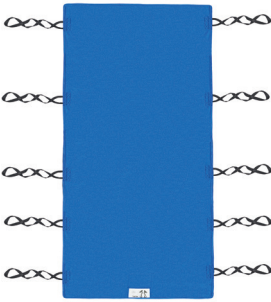

### Product Features

- IT material is designed to remain underneath patient for longer periods
- Attaches to ceiling and floor lifts w/ 2, 4, or 6 point hanger bar systems
- 4 elastic corner mattress straps help prevent sling from moving around on the bed
- Available in multiple colors
- Max Load of 454kg/1000lb with an option for 500kg/1100lb

### Options

#### Materials

- Open weave mesh–bathing and general use
- IT–**wicking material** w/ **woven antimicrobial**

#### Sizes

- Width Options: 45", 50", 60", or 66
- Length Options: 86", 100", or 118"

#### Attachments

- Color Ladder (CL)–industry standard
- F8 Numerical– numerical identification aids caregivers w/color blindness
- F8 Recoil (RC)- designed to prevent tripping with low support surfaces

## A-Limb Series

### Intended use:

To lift and hold limbs, eliminating the need to manually hold the limb for dressing changes, pericare, and/or placing pillows.



### Product Features

- The use of two limb slings will help with patients of size in holding limbs while on their side.
- Available in washable or wipeable
- Available in multiple colors
- Custom sizes available by request
- Max Load 150kg/330lb

### Options

#### **Materials**

- Tight weave mesh
- Urethane coated nylon

#### **Size**

- 8"W x 33"L

#### **Attachment Options**

- Color ladder (CL)–industry standard
- F8 Numerical– numerical identification aids caregivers w/color blindness
- Clip attachment option
- Wipeable loops (spray wipeable sling)
- Rainbow loop straps (SPU)

## A-TTurner Series

### Intended use:

To turn patient and hold patient onto their side.



### Product Features

- Fits into the lower back of patient
- Bariatric sizes available
- Available in washable or wipeable

### Options

#### Material

- Tight weave mesh
- Spacer mesh
- Coated polyester webbing - Wipeable
- Polyester webbing & Velcro - Washable
- Urethane coated nylon

#### Sizes - Max Weight / Length

- Regular: 150kg / 33"L"
- Bariatric: 300kg / 36.6"L"

#### Attachment Options

- Loop attachments

## A-Seat Series

### Intended use:

Seated transfers to and from bed, chair, toilet, commode and/or wheelchair.



### Product Features

- Available with or without head support
- Color coded to easily identify size
- Wipeable slings have tape sealed seams
- Amputee Strap option available
- Available in washable or wipeable
- Max load 454kg/1000 lbs
- Max load 500kg /1100 lbs (Bari line)

### Attachments

- Color Ladder (CL)–industry standard
- F8 Numerical– numerical identification aids caregivers w/color blindness
- Dual clip attachment option
- Wipeable loops- for spray wipeable sling

### Options

#### Materials

- Open weave mesh–bathing and general use
- Tight weave mesh–replaces linen on bed
- Urethane coated nylon–easy to wipe clean
- IT–**wicking material** w/ **woven antimicrobial** (Bari line only)

### Sizes (Color)/Recommended Weight Ranges

- Pediatric (Dino Print): <14kg
- Small (Red): 14-34kg
- Medium (Yellow ): 34-66kg
- Large (Green ): 66-100kg
- X-Large (Purple): 100-200kg
- XX Large (White ): 200-363kg
- Bari-X (Grey w purple trim ): 363- 500kg

## A-SitOnSling

### Intended use:

To provide full body support, without splitting the legs, while transferring to/from bed, chair, toilet, commode and wheelchair.



### Product Features

- Our Grey IT material is designed to remain underneath the patient for longer periods of time
- Sling binding is color coded to indicate size
- The sling offers support from the knees to the shoulders on patients
- Placement is accomplished by turning the patient side to side, positioning the sling at the bend of the knee.
- Max Load 454kg/1000lbs

### Options

#### Materials

- IT—**wicking material** w/ **woven antimicrobial**
- Polyester webbing and trim
- Coated polyester thread

#### Sizes (Trim Color) - Seat Width

- Medium (Yellow) - 30in/72cm
- Large (Green) - 32 in/81cm
- XL (Purple)- 35in/89cm
- XXL (White) - 39in/99cm



## A-Hygiene

### Intended use:

Seated transfers to and from bed, chair, toilet, commode and/or wheelchair with a belt to provide waist support.



### Product Features

- Color coded label to easily identify size
- Low back - support offered from knees to the shoulders of the patient
- Amputee Strap option
- Available in washable and SPU
- Max Load 454kg/1000 lbs

### Options

#### Materials

- IT–**wicking material** w/ **woven antimicrobial**

### Sizes/Recommended Weight Ranges

- Medium (Yellow): 34-66kg
- Large (Green): 66-100kg
- X-Large (Purple): 100-200kg
- XX Large (White): 200-363kg


### Attachment Options

- Color ladder (CL)–industry standard
- F8 Numerical– numerical identification aids caregivers w/color blindness

## A-SitStandWE Series

### Intended use:

Used with a powered mobile lift to assist a patient to a standing position.

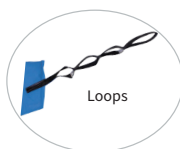

Sara Plus® is a registered trademark of Arjo Inc



### Product Features

- Color coded label indicates size
- Eliminates the need of having to purchase multiple units of the same size
- Inner safety belt that is adjustable and nonslip
- Easy to wipe with hospital grade disinfectant and will be ready immediately after cleaning
- Extender belt offered as an optional accessory (**sold separately**)
- Welded seams - eliminates the potential for fluid contamination
- Max Load 300kg /660lbs

### Options

#### Material

- Outer Belt- Herculete Fusion 9
- Inner Belt- Perfectex
- Wipeable coated polyester webbing on straps
- Coated polyester thread

#### Sizes / Inner Safety Belt Dimensions

- Small (Red) - 29"-46"
- Medium (Yellow) - 38"-55"
- Large (Green) - 45"-64"
- X-Large (Purple) - 55"-73"

#### Attachment Options

- Loop
- Clip
- Arjo Sara Plus® options

## Alpha Modalities

*Your Sling Source for Safe Patient Mobilization*

### A-WalkVSSWWL

#### **Intended use:**

To assist patient with ambulation and gait training.



#### Product Features

- Provide safety to patients who are at risk of falling while ambulating
- Multiple sizes identified by color of vest
- Design allows for toileting by releasing the clip on the groin straps
- Chemical and heat sealed seam
- Max load of all vests - 300kg/660lbs

#### Options

##### Material

- Coated polyester webbing and thread
- Metal Cobra buckles
- Urethane coated nylon
- Polyester spacer mesh

##### Sizes (Color) / Chest Measurement

- Pediatric (Blue) 20-26"/50-66cm
- Small (Red) 34-38"/86-97cm
- Medium (Yellow) 39-43"/99-109cm
- Large (Green) 44-48"/112-122cm
- Xlarge (Purple) 49-53"/123-135cm
- Extender Panel adds 16"/40cm


##### Attachment Options

- Wipeable Loops

## A-BariV Series

### Intended use:

To assist patients of size with ambulation and gait training.

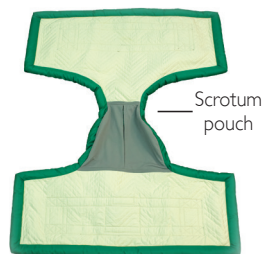

### Product Features

- Provide safety to patients of size who are at risk of falling while ambulating
- Multiple sizes identified by color of vest
- Comes in washable, wipeable, and SPU
- Cobra Buckles - either plastic or metal
- Washable sling has a scrotum pouch for patient comfort

### Options

#### Material

- Polyester
- Coated polyester webbing
- Spacer Mesh
- Urethane coated nylon
- Herculite Fusion III

#### Sizes(Color)/Max weight/Recommended Girth Range

- Small (Red) - 100kg / RGR 46"- 66"
- Medium (Yellow) - 200kg / RGR 54"-74"
- Large (Green) - 300kg / RGR 75"-94"
- Xlarge (Purple) - 400kg / RGR 89"-109"

#### Attachment Options

- Loop attachments

## A-Pannus

### Intended use:

To assist in moving and holding patient's panniculus away from the body for cleaning skin, wound checks, and/or dressing changes.



Access  
port for  
cleaning or  
exam

Optional  
accessory-  
O'Ring  
Headboard  
Straps



### Product Features

- Compatible with loop lift systems or can be used manually
- Comes with an extender belt kit which will extend the length of the bottom attachment loops
- Available in washable and SPU
- Max Load 454kg / 1000lbs

### Options

#### Materials

- IT—**wicking material** w/ **woven antimicrobial**
- Soft inner wicking liner
- Polyester webbing

### Optional Accessory - O'Ring Headboard Straps

Helps provide extra lift for patients with a deep panniculus fold and eliminates the need for staff to exert extra force gripping the straps in order to pull panniculus up and back toward headboard - **sold separately**

## A-WedgeWE Series

### **Intended use:**

To keep patient on their side or to prevent sliding down in bed.



### Product Features

- Designed with black non skid surface
- Place between the bed and patient's linen to keep patient on their side.
- The white wipeable handle allows you to move and position the wedge easily.
- The wedge is sealed to allow for easy cleaning by wiping down with hospital approved disinfectant

### Options

#### Material

- 100% polyester
- Herculite Fusion III
- Waterproof
- Chemical seam sealers

#### Fill Options

- Medical grade foam - firm
- Spun polyester - soft

#### Sizes

- 16"/40.6cm Length
- 23"/58.4cm

## A-BedLadderSW

### **Intended use:**

To help patients sit themselves up in bed, assist in moving legs in/out of bed or vehicle and general repositioning on a flat surface.



### Product Features

- MRI Compatible
- Soft, pliable grips
- In some cases, the BedLadder can take the place of a trapeze
- Max Load - 272kg/600lbs

### Options

#### *Material*

- Coated polyester webbing and thread
- Herculite Fusion III
- Closed cell foam
- Chemical seam sealer

#### *Size*

- 12"/37cm width 63"/160cm



## SPU Series

### Intended use:

Single Patient Use slings that offer the same functions as our TSL, Seat, Limb, Hygiene, BariVest, and Pannus slings.



### Product Features

- An option for facilities who have challenges with a reusable sling management process
- Variety of sling styles available
- Label that is clearly visible to write specific patient ID
- Lightweight and individually packaged in clear sleeves
- Can be spot cleaned with hot water if soiled by patient
- Slings are packaged 10 per case
- **BariKit** - All-in-one pack includes 1 of each of the following SPU slings: Repositioning, Limb, and a BariVest - Max Load 454kg

### Options/Sizes

#### Flat Repo Sling (TSL)

- 45"X86" / 114x218cm Max Load 454kg
- 55"X86" / 140x218cm Max Load 500kg

#### Seated Sling

- M/L Combo Max Load 100kg
- X/XX Combo Max Load 363kg

**Limb Sling** - Universal Size Max Load 150kg

#### Hygiene Sling

- M/L Combo Max Load 363kg
- X/XX Combo Max Load 363kg

**BariVest** - Max Load 454kg

- Recommended Girth Range 75"-94"

#### PannusSPU

- Max Load 454kg



## ECO Series

### Intended use:

Environmentally friendly slings that can be laundered for additional usage, helping to reduce your carbon footprint.



### Product Features

- Synthetic breathable material that is strong and comfortable.
- More cost effective than traditional SPU slings - our quality fabric can be rewashed up to 50 times for additional usage without the worry that the fabric will wear out.
- Variety of sling styles available
- Slings are packaged 10 per case
- 90-day limited warranty

### Options/Sizes

#### Flat Repo Sling (TSL)

- 45"X86" / 114x218cm Max Load 454kg
- 55"x86" / 140x218cm Max Load 500kg

#### Seated Sling-comes standard w/amputee straps


- M/L Combo Max Load 100kg
- X/XX Combo Max Load 363kg

#### Limb Sling

- Universal Size Max Load 150kg

#### Hygiene Sling

- M/L Combo Max Load 363kg
- X/XX Combo Max Load 363kg

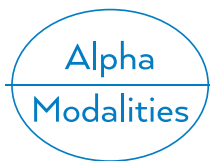

*Your Sling Source for Safe Patient Mobilization*

## Cleaning Instructions

Alpha Modalities LLC Washable, Wipeable, ECO, SPU and Herculite Fabric products cleaning instructions are listed below. Failure to clean per our guidelines will void product warranty.

### Washable and ECO products



- Commercial laundry systems with maximum load 850lbs - **tunnel washer voids warranty**
- Oxygen based bleach solutions only - **chlorine based solution voids warranty**
- Fresh water rinse cycle of minimum eight (8) minutes.
- Wash temp not to exceed 185F / 85 C
- Dry temp not to exceed 145F / 74C
- ECO Line Products: Only wash for 90 days after first use date.

### Wipeable(SW) products



- **DO NOT LAUNDER** - Wipe with hospital approved facility disinfectant for "Non-Critical Devices", as per FDA guidelines or 10% bleach solution.
- **DO NOT soak** in solution of chlorine based bleach solution.
- For blood stains: if the blood is fresh and has not dried, **cold water** must be used to clean. **DO NOT USE HOT WATER** as it sets the stains to iron color from the blood. If it has dried, a spot clean of the stain of 50% Hydrogen Peroxide and cold water can be used.

### SPU products



- **DO NOT LAUNDER** - may be spot cleaned with hot water if soiled by patient.


### Herculite Fabric Items - (SitStandWE & WedgeWE Series welded products)



- **DO NOT LAUNDER** - For soils or stains, use neutral soap suds and lukewarm water. **DO NOT use harsh cleansers, solvents or detergents.** For hard to clean spots, use a soft bristle brush and gentle agitation on troublesome spots or stains. Pre-soak heavy, dried-on soil.
- Disinfecting - Quaternary and quaternary/Isopropyl disinfectants are recommended. **DO NOT use phenolic disinfectants.** May be cleaned with a 1:10 dilution of household bleaches containing 5.25% sodium hypochlorite as recommended by the CDC in Atlanta, GA. There is no harmful effect when used in this dilution ratio.
- Disinfectants applied at full concentration or in highly concentrated solutions may damage the fabric.
- Iodophor type disinfectants may result in staining.

## **DO NOT DRY CLEAN OR IRON ANY PRODUCTS**

Always inspect sling before use. If damaged or torn, please contact our Customer Support Center at [www.alphamodalities.com](http://www.alphamodalities.com).

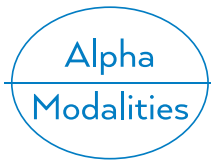

*Your Sling Source for Safe Patient Mobilization*

## Warranty

Our Unlimited 3 Year Warranty begins when you first use the sling or batch tag date, depending on which is easier to identify. The warranty is unlimited for the first three years and covers manufacturing defects, fabric wear and tear and loose threads. The warranty will be void if it is determined that washable slings have been cleaned with a chlorine based bleach solution or that repairs/ modifications have been made to the slings. Wipeable slings are not to be laundered in commercial system as this will void the warranty.

We will repair or replace any sling found to be defective in manufacturing ***at no charge if it is under warranty.*** Washable slings out of warranty but repairable will be charged a fee for the repairs (upon client authorization). Sling must be returned to us clean, freight prepaid and attached to a Return Authorization Number (RMA) which can be obtained by visiting our Customer Support Center at [www.alphamodalities.com](http://www.alphamodalities.com).

**ALL PRODUCTS ARE PROP 65 COMPLIANT.**

**ALL PRODUCTS ARE LATEX FREE.**

**ALL PRODUCTS ARE MADE IN THE USA.**



