


Your Sling Source for Safe Patient Mobilization

1111 South 344th Street, Suite 205, Federal Way, WA 98003
 800.273.5749 • info@alphamodalities.com

A-TSLECO (4pt hanger bar)

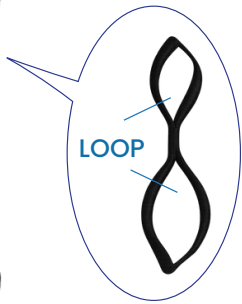

Part Number	Size	Max Load
A-TSLECO	45"W x 86" L	454 kg / 1000 lbs
A-TSLECOB	55" W x 86" L	500kg / 1100 lbs


- If needed, a chuck may be placed between patient and sling
- Label should be at patient's foot.
- Pillow may be placed between legs for comfort.

NOTE: The attachment configurations shown are a *general guideline*. Please adhere to your hospital's policy and use your clinical judgment as to what best meets your patient's needs.

USING THE TSL TO TURN PATIENT

Attach two straps closest to **patient's midsection**: close to patient's **shoulder** and **hip**



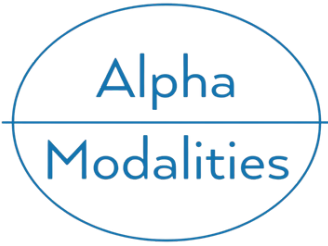

Your Sling Source for Safe Patient Mobilization

1111 South 344th Street, Suite 205, Federal Way, WA 98003
800.273.5749 • info@alphamodalities.com

**A-TSLECO
(4pt hanger bar)**

SUPINE TRANSFER

Generally use the same length loop all the way down



SEMI-RECLINED TRANSFER

Generally use short loop for strap 1 and the longer loop for the rest of the straps. Attach strap 5 if patient tolerates leg hyperextension - otherwise, leave strap and the longer loop for the rest of the straps



Cleaning Instructions

- Max Load 850 lbs
- Oxygen based solutions only
- Wash temp not to exceed 185 F / 85 C
- Dry temp not to exceed 145 F / 74C
- 90 day warranty

DISCLAIMERS

- Discontinue use if loose stitching or tears are visible.
- Do not use with an uncleared C-Spine.