

# Wipeable SPHM Solutions



*Designed for Compassion  
Engineered for Strength*



Supporting The Journey to Recovery Since 1996

# 30

YEARS



# What does wipeable mean?



Alpha Modalities created the wipeable line in order to bridge the gap between washable and single patient use SPHM products. Wipeable products are made with non-porous fabrics, so they can be cleaned with most hospital approved disinfectants currently used to clean other hospital equipment. Please refer to 'Cleaning Instructions' for more details.



# Where are wipeable slings used?



Wipeable products are used when laundering is unavailable or single patient use products are not feasible from a cost or environmental perspective. While they are appropriate for most healthcare settings, wipeable products are most commonly used in these care areas.

*Medical-Surgical  
Critical Care  
Bariatric Care*



*Rehabilitation  
Radiology  
Ancillary Departments*





*Outpatient or  
Ambulatory Care  
Clinics*



# Designed to Wipe Clean

## Built to Last

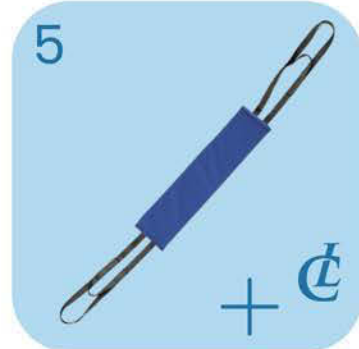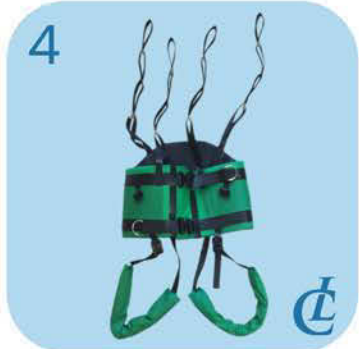

- 1. Wedges
- 2. Seated Sling
- 3. BariVest
- 4. WalkVest
- 5. Limb
- 6. Twin Turner
- 7. SitStand
- 8. Bed Ladder



All wipeable solutions are USA made & backed by our industry leading warranty




+ Clip and loop compatible options available.  
 ℄ Mobile and overhead lift compatible. Intended use with 2 point, 4 point, and 6 point hanger bars.



**Foam Wedge**  
Firm Support



**Spun Polyester Wedge**  
Soft Support

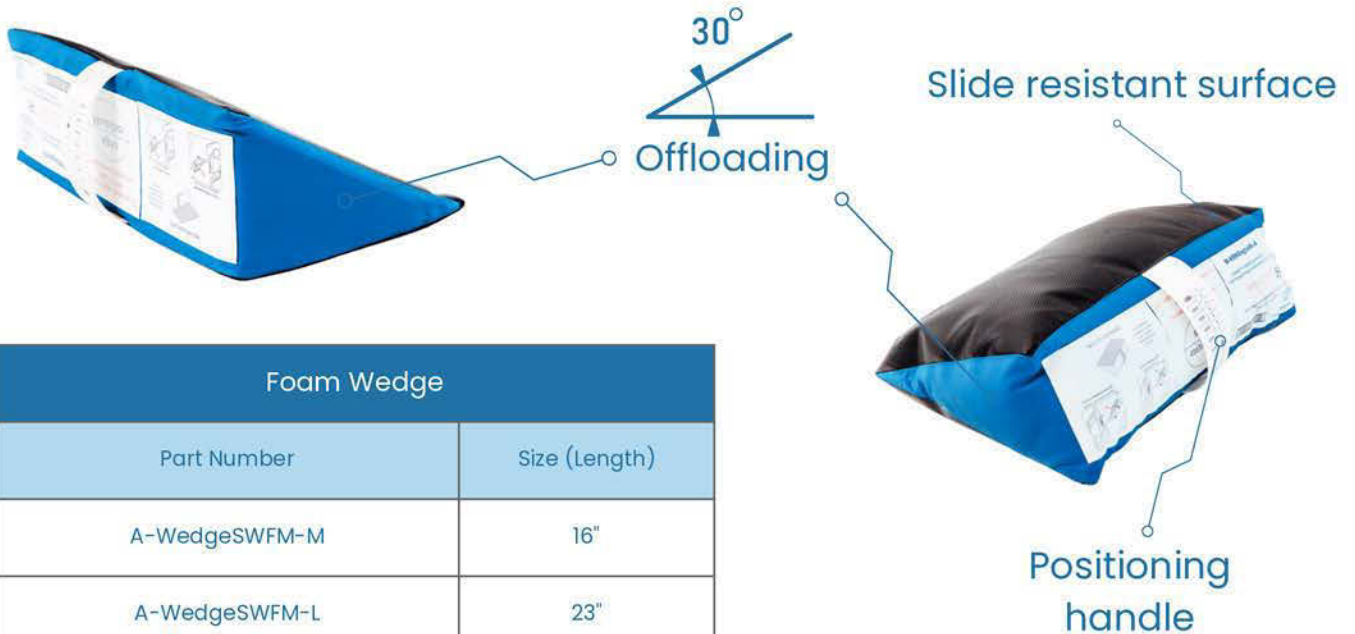

Foam Wedge	
Part Number	Size (Length)
A-WedgeSWFM-M	16"
A-WedgeSWFM-L	23"
Spun Polyester Wedge	
A-WedgeSWBN-M	16"
A-WedgeSWBN-L	23"




# Alpha Modalities

## Seat Wipeable Series



**100%  
Non-Porous  
Latex Free Fabric**

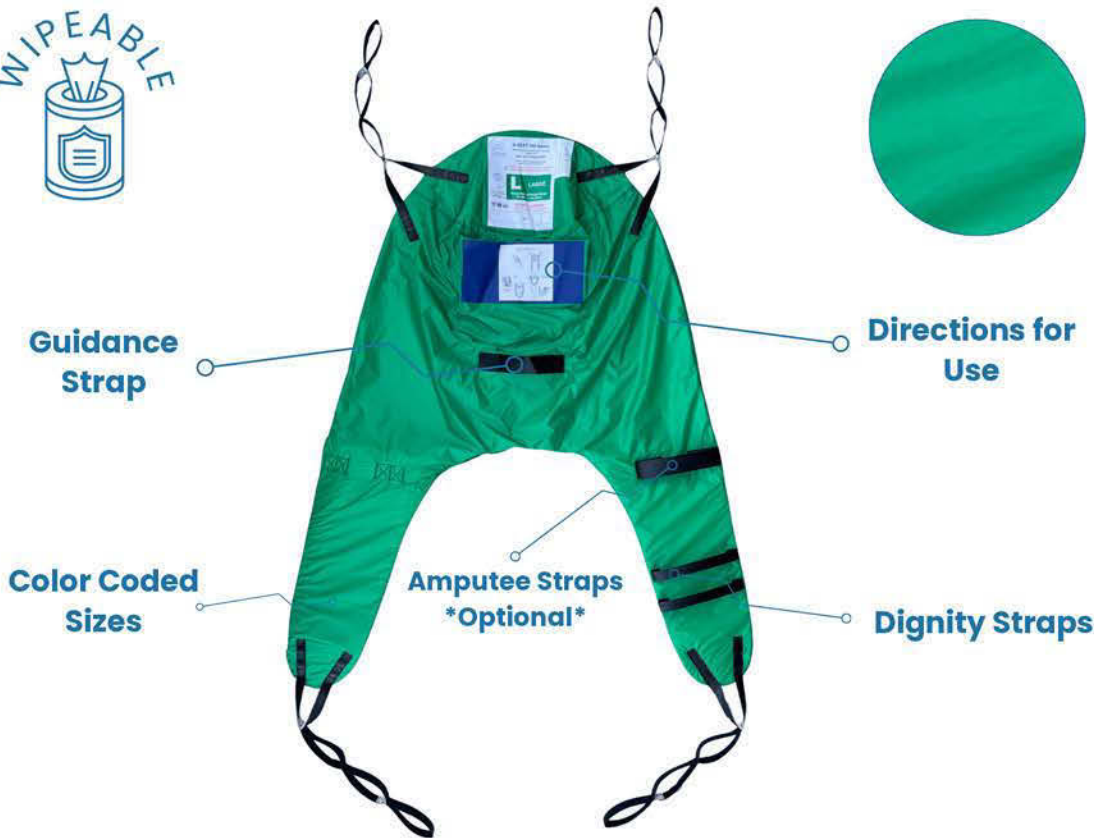

Seat Sling			
Color Coded Size	Part Number	Recommended Weight Range (lbs)	Safe Working Load
Small	A-SeatSWHSWL-S (Loop) A-SeatSWHSDC-S (Clip)	31 - 75	1000
Medium	A-SeatSWHSWL-M (Loop) A-SeatSWHSDC-M (Clip)	75 - 145	
Large	A-SeatSWHSWL-L (Loop) A-SeatSWHSDC-L (Clip)	145 - 220	
X-Large	A-SeatSWHSWL-X (Loop) A-SeatSWHSDC-X (Clip)	220 - 440	
XX- Large	A-SeatSWHSWL-XX (Loop) A-SeatSWHSDC-XX (Clip)	440 - 800	

### Attachment Options



**Loop**



**Dual Clip**



Alpha  
Modalities

**BariVest**  
Wipeable Series



Color coded sizing for accurate selection


Quick release, cobra buckles for simple application and removal



Padded, spacer mesh for comfort

BariVest			
Color Coded Size	Part Number	Recommended Waist Circumference (inches)	Safe Working Load (pounds)
X-Small	A-BariVSWWL-50	24" - 38"	880
Small	A-BariVSWWL-100	46" - 66"	
Medium	A-BariVSWWL-200	54" - 74"	
Large	A-BariVSWWL-300	75" - 94"	
X-Large	A-BariVSWWL-400	89" - 109"	



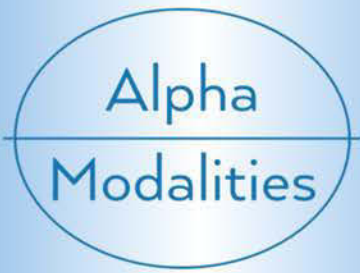

# WalkVest

Wipeable Series



Cobra buckle secure support




Size adaptable around waist and leg straps

Quick release leg strap for toileting



WalkVest			
Color Coded Size	Part Number	Recommended Chest Circumference Range (inches)	Safe Working Load (pounds)
X-Small	A-WalkVSSWWL-Ped	20" - 26"	660
Small	A-WalkVSSWWL-S	32" - 39"	
Medium	A-WalkVSSWWL-M	40" - 44"	
Large	A-WalkVSSWWL-L	42" - 47"	
X-Large	A-WalkVSSWWL-X	49" - 54"	
	Part Number	Accessory - Extender Panel	
Same as specified WalkVest size	A-WalkVSSW-Extndr	Adds 16 inches, available on all sizes	



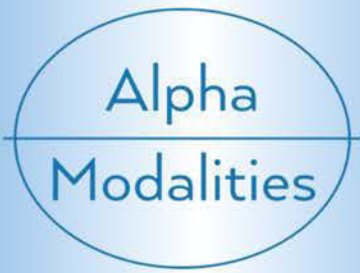

# Limb

Wipeable Series



Dual clip or dual loop customizable options



Padded for enhanced patient comfort



Unilateral Extremity Support



Sidelying Pericare Assist




Bilateral Extremity Positioning

Limb		
Part Number	Loop or Clip Compatible	Safe Working Load (pounds)
A-LimbSWWL	Loop	330
A-LimbSWDC	Clip	



Quick Reference Guide

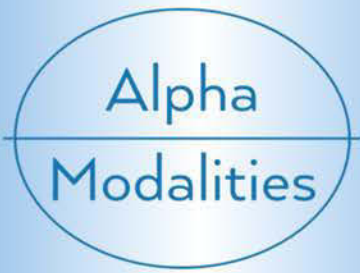


# Twin Turner

Wipeable Series



Multi-layer padding for enhanced comfort

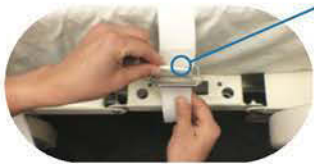

Bed frame attachment buckle

Contoured design for optimal patient care




**Standard**



**Bariatric**

Twin Turner			
Part Number	Size	Length (inches)	Safe Working Load (pounds)
A-TTurnerSW	Standard	33"	330
A-TTurnerBSW	Bariatric	36.6"	660



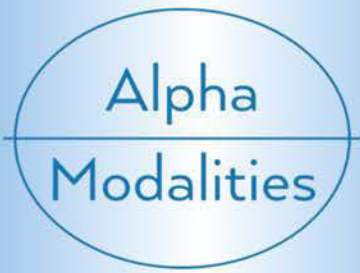

# SitStand Belts


Wipeable Series



## SitStand Belts


	Loop	Clip	Sara Plus®
--	------	------	------------

Size	Inner Safety Belt Range (inches)	Part Number		
Small	29" - 46"	A-SitStandWEWL-S	A-SitStandWEDC-S	A-SitStandWESPLS-S
Medium	38" - 55"	A-SitStandWEWL-M	A-SitStandWEDC-M	A-SitStandWESPLS-M
Large	45" - 64"	A-SitStandWEWL-L	A-SitStandWEDC-L	A-SitStandWESPLS-L
XLarge	55" - 73"	A-SitStandWEWL-X	A-SitStandWEDC-X	A-SitStandWESPLS-X
Accessory - SitStand Belt Extender		Part Number		
Adds additional *** inches, only compatible with loop combinations		A-SitStandSW-Extndr		

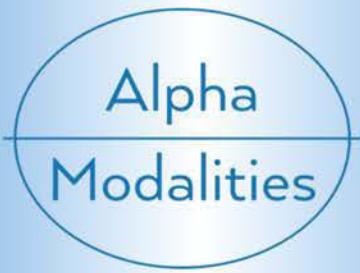

# Bed Ladder

Wipeable Series

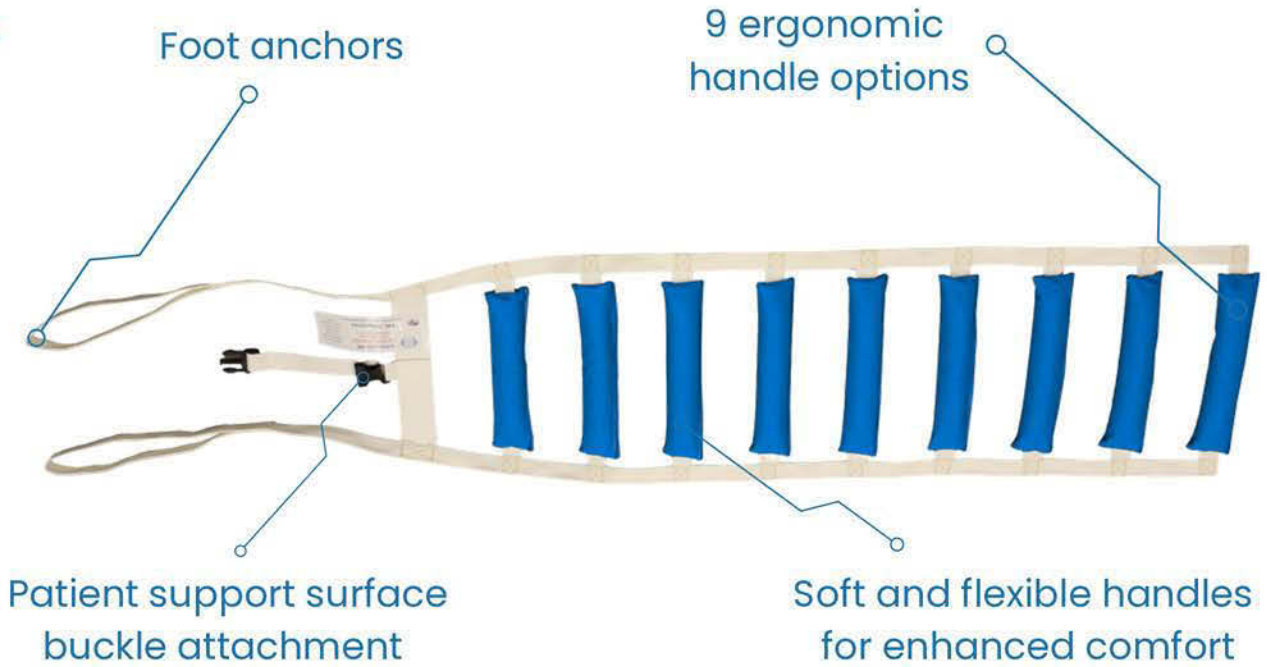

Bed  
Mobility Aid



Exam Table  
Positioning Support



Limb  
Assist

**\*Do not leave unattended with patient \***

Bed Ladder		
Part Number	Size (inches)	Safe Working Load (pounds)
A-BedLadderSW	Universal: 63 X 12	600





# Cleaning Instructions

## Wipeable Series



### Spray Wipeable Products

**Seated Sling ~ BariVest ~ WalkVest ~ Limb Sling ~ Twin Turner ~ Tube Slider ~ Bed Ladder**

**DO NOT LAUNDRER** - Wipe with hospital approved facility disinfectant for "Non-Critical Devices", as per FDA guidelines or 10% bleach solution. Do not dry clean or iron.

DO NOT soak in solution of chlorine based bleach solution.

For blood stains: if the blood is fresh and has not dried, cold water must be used to clean.

DO NOT USE HOT WATER as it sets the stains to iron color from the blood. If it has dried, a spot clean of the stain of 50% Hydrogen Peroxide and cold water can be used.

### Herculite Fabric Items

**SitStand Belts ~ Wedges**

**DO NOT LAUNDRER** - For soils or stains, use neutral soap suds and lukewarm water. Do not dry clean or iron.

DO NOT use harsh cleansers, solvents or detergents. For hard to clean spots, use a soft bristle brush and gentle agitation on troublesome spots or stains. Pre-soak heavy, dried-on soil.

Disinfecting - Quaternary and quaternary/Isopropyl disinfectants are recommended.

DO NOT use phenolic disinfectants. Iodophor type disinfectants may result in staining.

May be cleaned with a 1:10 dilution of household bleaches containing 5.25% sodium hypochlorite as recommended by the CDC. There is no harmful effect when used in this dilution ratio.

Disinfectants applied at full concentration or in highly concentrated solutions may damage the fabric.



Failure to clean per guidelines will void product warranty. Consult with your Alpha Modalities Account Manager for any questions.

